

USER MANUAL

RTP[®] Pathogen Kit

for purification of bacterial DNA and viral DNA/RNA
from clinically relevant starting materials (e.g. serum, plasma, blood, stool suspension,
bacterial enrichment culture, tissue)

Instruction

RTP® Pathogen Kit

The RTP® Pathogen Kit is the optimal tool for simultaneous isolation of high quality bacterial and viral DNA as well as viral RNA from human and animal serum and plasma samples, cell culture supernatants and other cell-free body fluids, like urine as well as from swabs or tissue biopsies for *in-vitro* diagnostic purposes using spin-filter format.

Fresh or frozen plasma or serum from blood treated with anticoagulants like EDTA or citrate, *but not with heparin* can be used.

The customer convenient well established RTP® technology using prefilled Extraction Tubes simplifies the process handling, reduces the handling steps with infectious material and allows a process monitoring.

Due to the high purity, the isolated viral DNA / RNA and bacterial DNA is ready to use for a broad panel of downstream applications or can be stored at –80 °C for subsequent use.

The kit is neither validated for the isolation of genomic DNA from stool sample, parasites or the purification of total RNA from clinical samples.



Compliance with EU Directive 98/79/EC on *in vitro* medical devices.

Not for *in-vitro* diagnostic use in countries where the EU Directive 98/79/EC on *in vitro* medical devices is not recognized.

Trademarks: Invisorb®, RTP®, InviMag®, Eppendorf®. Registered marks, trademarks, etc. used in this document, even when not specifically marked as such, are not to be considered unprotected by law.

The Invisorb® technology is covered by patents and patent applications: US 6,110,363, US 6,043,354, US 6,037,465, EP 0880535, WO 9728171, WO 9534569, EP 0765335, DE 19506887, DE 10041825.2, WO 0034463.

Invisorb®, RTP® and InviMag® are registered trademarks of Invitek Molecular GmbH.

The PCR process is covered by US Patents 4,683,195, and 4,683,202 and foreign equivalents owned by Hoffmann-La Roche AG.

© 2020 Invitek Molecular, all rights reserved.

Contents

Kit contents of RTP® Pathogen Kit	3
Symbols	4
Storage	4
Quality control and product warranty	4
Intended use	5
Product use limitation	5
Safety information	6
Product characteristic of the RTP® Pathogen Kit	7
Internal control (IC)/ Extraction control	9
Principle	9
Procedure	9
Important notes	10
Important points before starting a protocol	10
Important indications	10
Yield and quality of pathogen DNA/ RNA	11
Preparing buffers	11
Equipment and reagents to be supplied by user	12
Sampling and storage of starting material	12
Volume adjustment of sample material	14
Preparation of starting material	14
Instructions	16
Scheme	17
<i>Protocol:</i> Simultaneous isolation of total nucleic acids (DNA and RNA) from all liquid samples	18
Troubleshooting	19
General notes on handling DNA	20
General notes on handling RNA	21
Ordering information	22

Kit contents of RTP® Pathogen Kit

	50 preps	250 preps
Catalogue No.	1040500200	1040500300
Extraction Tube	50	5 x 50
Resuspension Buffer R	30 ml	150 ml
Binding Solution (fill with 99.7% Isopropanol)	empty bottle (final volume 30 ml)	empty bottle (final volume 120 ml)
Wash Buffer R1	20 ml (final volume 40 ml)	80 ml (final volume 160 ml)
Wash Buffer R2	12 ml (final volume 60 ml)	50 ml (final volume 250 ml)
Elution Buffer R	15 ml	60 ml
RTA Spin Filter Set	50	5 x 50
RTA Receiver Tubes	3 x 50	15 x 50
Elution Tube	50	5 x 50
Manual	1	1
Initial steps	<p>Fill 30 ml 99.7% Isopropanol (molecular biologic grade) into the empty bottle</p> <p>Add 20 ml of 96 – 100 % ethanol to the bottle Wash Buffer R1, mix thoroughly and always keep the bottle firmly closed!</p> <p>Add 48 ml of 96 – 100 % ethanol to the bottle Wash Buffer R2, mix thoroughly and always keep the bottle firmly closed!</p>	<p>Fill 120 ml 99.7% Isopropanol (molecular biologic grade) into the empty bottle</p> <p>Add 80 ml of 96 – 100 % ethanol to the bottle Wash Buffer R1, mix thoroughly and always keep the bottle firmly closed</p> <p>Add 200 ml of 96 – 100 % ethanol to the bottle Wash Buffer R2, mix thoroughly and always keep the bottle firmly closed!</p>

Symbols

	Manufacturer	
	Lot number	Attention: Do not combine components of different kits, unless the lot numbers are identical!
	Catalogue number	
	Expiry date	
	Consult operating instructions	
	Temperature limitation	
	Do not reuse	
	Humidity limitation	

Storage

All buffers and kit contents of the **RTP® Pathogen Kit** including the **Extraction Tube** (incl. Lysis Buffer, Proteinase K, Lysozyme, Carrier RNA and Internal Control DNA) should be stored at room temperature and are stable for at least 12 months. Store the Kit especially the Extraction Tubes in a dry environment, the Extraction Tubes must be protected from humidity.

Room temperature (RT) is defined as range from 15 - 30°C.

Before every use, make sure that all components have room temperature. If there are any precipitates within the provided solutions solve these precipitates by warming carefully (up to 30°C).

Wash Buffers charged with ethanol should be appropriately sealed and stored at room temperature.

Binding Solution charged with isopropanol should be appropriately sealed and stored at room temperature.

Quality control and product warranty

Invitek Molecular warrants the correct function of the **RTP® Pathogen Kit** for applications as described in this manual. Purchaser must determine the suitability of the Product for its particular use. Should any Product fail to perform the applications as described in the manual, Invitek Molecular will check the lot and if Invitek Molecular investigates a problem in the lot, Invitek Molecular will replace the Product free of charge.

Invitek Molecular reserves the right to change, alter, or modify any Product to enhance its performance and design at any time.

In accordance with Invitek Molecular's EN ISO 13485 certified Quality Management System the performance of all components of the **RTP® Pathogen Kit** have been tested separately against predetermined specifications routinely on lot-to-lot to ensure consistent product quality.

If you have any questions or problems regarding any aspects of **RTP® Pathogen Kit** or other Invitek Molecular products, please do not hesitate to contact us. A copy of Invitek Molecular's terms and conditions can be obtained upon request or are presented at the Invitek Molecular webpage www.invitek-molecular.com.

For technical support or further information, please contact:

Email: techsupport@invitek-molecular.com or contact your local distributor.

Intended use

The RTP® Pathogen Kit is the ideal tool for reliable and fast simultaneous isolation of high quality bacterial and viral DNA as well as viral RNA from fresh or frozen human or mammalian serum, plasma, cell culture supernatants and other cell free body fluids as well, swabs or tissue biopsies as well as for efficient isolation of high quality high bacterial DNA from bacterial species, tissue samples, urine, paper points, swabs and water using spin-filter format.

For reproducible high yields an appropriate sample storage and quick operation under the rules for RNA and DNA operation is essential. The purified viral DNA and/or RNA and bacterial DNA is ready to use for *in-vitro* diagnostic analysis.

The isolation protocols as well as all buffers are optimized to provide high yield and purity of the extracted viral & bacterial nucleic acids. The procedure requires minimal interaction by the user, allowing safe handling of potentially infectious samples.

THE PRODUCT IS INDENTED FOR USE BY PROFESSIONAL USERS ONLY, SUCH AS TECHNICIANS, PHYSICIANS AND BIOLOGISTS TRAINED IN MOLECULAR BIOLOGICAL TECHNIQUES. It is designed to be used with any downstream application employing enzymatic amplification or other enzymatic modifications of DNA/ RNA followed by signal detection or amplification. Any diagnostic results generated by using the sample preparation procedure in conjunction with any downstream diagnostic assay should be interpreted with regard to other clinical or laboratory findings.

To minimize irregularities in diagnostic results, adequate controls for downstream applications should be used.

The kit complies with EU Directive 98/79/EC on in vitro medical devices. However, it is not for in-vitro diagnostic use in countries where the EU Directive 98/79/EC on in vitro medical devices is not recognized.

Product use limitation

The kit is neither validated for the isolation of vertebrate or parasite genomic DNA or total RNA, for the purification of pathogen NA from heparin, EDTA or citrate stabilized blood nor for the purification of pathogen NA from Heparin stabilized serum or plasma.

The included chemicals are only useable once. Differing of starting material or flow trace may lead to inoperability; therefore, neither a warranty nor guarantee in this case will be given, neither implied nor express. The user is responsible to validate the performance of the Invitek Molecular Product for any particular use. Invitek Molecular does not provide for validation of performance characteristics of the Product with respect to specific applications.

Invitek Molecular products may be used e.g. in clinical diagnostic laboratory systems under following conditions:

- If used in the US, based on the condition that the complete diagnostic system of the laboratory has been validated pursuant to CLIA' 88 regulations.
- For other countries based on the condition that the laboratory has been validated pursuant to equivalents according to the respective legal basis.

All Products sold by Invitek Molecular are subject to extensive quality control procedures (according to EN ISO 13485) and are warranted to perform as described herein. Any problems, incidents or defects shall be reported to Invitek Molecular immediately upon detection thereof.

The chemicals and the plastic parts are for laboratory use only; they must be stored in the laboratory and must not be used for purposes other than intended.

The Product with its contents is unfit for consumption.

Safety information

When and while working with chemicals, always wear a suitable lab coat, disposable gloves, and protective goggles!

Avoid skin contact! Adhere to the legal requirements for working with biological material!

For more information, please consult the appropriate material safety data sheets (MSDS). These are available online in convenient and compact PDF format at www.invitek-molecular.com for each Invitek Molecular Product and its components. If buffer bottles are damaged or leaking, WEAR GLOVES, AND PROTECTIVE GOGGLES when discarding the bottles in order to avoid any injuries.

Invitek Molecular has not tested the liquid waste generated by the RTP® Pathogen Kit procedures for residual infectious materials. Contamination of the liquid waste with residual infectious materials is highly unlikely, but cannot be excluded completely. Therefore, liquid waste must be considered infectious and be handled and discarded according to local safety regulations.

European Community risk and safety phrases for the components of the RTP® Pathogen Kit to which they apply are listed below as follows:

Extraction Tube



Warning

H302-H315-H319-H335-H411-P280-P305+P351+P338-EUH208

Wash Buffer R1



Warning

H302-H332-H412-P280-P305+P351+P338-EUH032

H302: Harmful if swallowed.

H315: Causes skin irritation.

H319: Causes serious eye irritation.

H332: Harmful if inhaled.

H335: May cause respiratory irritation.

H411: Toxic to aquatic life with long lasting effects.

H412: Harmful to aquatic life with long lasting effects.

P280: Wear protective gloves/protective clothing/eye protection/face protection.

P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

EUH208: Contains Proteinase, Tritirachium album-Serine. May produce an allergic reaction.

EUH032: Contact with acids liberates very toxic gas.

Emergency medical information can be obtained 24 hours a day from infotrac:

outside of USA: 1 – 352 – 323 – 3500

in USA: 1 – 800 – 535 – 5053

Product characteristic of the RTP® Pathogen Kit

Starting material	Yield	Time
<ul style="list-style-type: none"> - up to 200 µl cell-free body fluids, cell culture supernatants - tracheal secretes, BAL - up to 400 µl rinse liquid from swab - max. 10 mg tissue sample - stool samples - bacteria cultures 	<p>depends on the sample (storage and source)</p> <p>Note: The added Carrier RNA will account for most of the eluted NA. Quantitative (RT)-PCR is recommended for determination of the viral RNA or DNA yield.</p>	20 minutes

The **RTP® Pathogen Kit** provides a fast and efficient way for reliable simultaneous isolation of high-quality viral DNA / RNA and bacterial DNA from a diverse range of starting material. The procedures are suitable for use with plasma or serum; either can contain citrate or EDTA (no heparin), tissue samples, urine, paper points, stool samples and swabs. Samples can be fresh or frozen, provided they have not been frozen and thawed more than ones.

The amount of purified DNA and/ or RNA in the **RTP® Pathogen Kit** procedures depend on the sample type, sample source, transport, storage, age and the virus titer.

The kits use the patented RTP® technology, whose special feature is the **Extraction Tube** containing already pre-formulated solid lysis reagent, lytic enzyme, Proteinase K and Carrier Nucleic Acids. Only a minimum of manual work by the user is necessary. The procedure is designed to avoid sample-to-sample cross-contaminations and allow safe handling of potentially infectious samples.

Using the **RTP® Pathogen Kit**, all types of samples are transferred into the **Extraction Tubes** together with a specially designed **Resuspension Buffer** to adjust a final sample volume of 400 µl. The prefilled buffer and enzymes lyse the samples, stabilize the nucleic acids and enhance the viral and bacterial DNA and/ or RNA adsorption to the membrane in the RTA Spin Filter. Contaminants are removed by repeated wash steps and the purified nucleic acids can be eluted in a small volume of **Elution Buffer**.

In addition to the rigorous lysis procedure, simple pre-treating steps have been introduced, ideally for purification of DNA, especially from Gram-positive bacteria of different sources.

The advantage of the kits results from the simultaneous isolation of nucleic acids from DNA and RNA viruses as well as from bacteria species. That enables the user to test the sample for all kinds of nucleic acids from viruses and bacteria, which are of interest after one preparation. High extraction efficiency and detection sensitivities will be realized. No phenol chloroform extraction or β-Mercaptoethanol is required. The “hands-on time“ necessary for the whole procedure is reduced to a minimum. All kit components can be stored at room temperature.

Yield and quality of the isolated nucleic acids are suitable for any molecular diagnostic detection system. The diagnostic tests should be performed according to manufacturer's specifications. Due to the high purity, the isolated viral DNA/ RNA and bacterial DNA is ready to use for a broad panel of downstream applications or can be stored at -80°C for subsequent use.

- (RT)-PCR
- real-time PCR (quantitative (RT)-PCR, like TaqMan und LightCycler technology)
- cDNA synthesis
- mircoarray-application
- RFLP-Analysis

Note: *Systems isolating simultaneously DNA and RNA using buffers adapted for the binding of DNA and RNA, but the optimal binding conditions of RNA and DNA are different, so that such solutions can show a little reduction in sensitivity in comparison to kits optimized to one kind of nucleic acid isolation.*

Internal control (IC)/ Extraction control

Internal Controls (IC) from the PCR assay provider can be used as extraction controls if the fragments are longer than 100 bp, the recovery rate is depending on the length of the fragments, so the best internal controls have a length comparable to the target nucleic acids. If nucleic acids are used for this purpose, they have to be added after finalization of the lysis step.

Attention: Do not add directly the Internal Controls to the sample!

Principle

The RTP® Pathogen Kit procedure comprises following steps:

1. lysis at different temperatures
2. adjustment of the binding conditions
3. binding the nucleic acids in the lysate to the membrane of a RTA Spin Filter
4. washing of the membrane and elimination of contaminants and ethanol
5. elution of the nucleic acids

Procedure

Bacteria must be cultivated under special conditions and an aliquot of the bacteria suspension will be used to win a bacteria pellet by centrifugation at high speed for 5 min. The supernatant will be removed.

Lysis

Samples are lysed under non-chaotropic conditions at different, elevated temperature and continuously shaking. Lysis is performed in the presence of Lysozyme to break the cell wall of the bacteria, a **Lysis Buffer** and **Proteinase K** to digest the proteins. All is provided prefilled in the **Extraction Tube**. Unlysed sample parts should be removed before the binding step. Due to the strong denaturing lysis conditions RNases and DNases are inactivated simultaneously.

The addition of **Carrier RNA** (provided in the Extraction Tube) is necessary for the enhancement of viral DNA/ RNA recovery so a very small number of viral DNA/ RNA molecules will also be purified. **Carrier RNA** also stabilizes nucleic acids in samples with very small nucleic acid concentrations.

Binding nucleic acids

After adding **Binding Solution** to adjust optimal binding conditions the lysate will be applied onto the RTA Spin Filter and the nucleic acids are bound to the surface of the RTA Filter membrane as the lysate is drawn through by centrifugation.

Removing residual contaminants

Contaminants are efficiently washed away using **Wash Buffer R1** and **R2**, while the nucleic acids remain bound to the membrane of the RTA Spin Filter.

Elution

High quality viral DNA/ RNA and genomic DNA is eluted from the membrane using **Elution Buffer R** or RNase free water. Eluting twice each with 100 µl leads to little increase of DNA/RNA yield. Usage small elution volumes may raise the DNA/RNA concentration. Elution volumes should be at least 40 µl. The volume of eluate recovered may be up to 5 µl less than the volume of elution buffer applied to the RTA Spin Filter. The volume of eluate recovered depends on the nature of the sample.

The eluted DNA/ RNA is ready to use in different subsequent applications.

Important notes

Important points before starting a protocol

Immediately upon receipt of the Product, inspect the Product and its components as well as the package for any apparent damages, correct quantities and quality. If there are any unconformities you have to notify Invitek Molecular in writing with immediate effect upon inspection thereof. If buffer bottles are damaged, contact the Invitek Molecular Technical Services or your local distributor. In case of liquid spillage, refer to “Safety Information” (see page 6). Do not use damaged kit components, since their use may lead to poor kit performance.

1. Always change pipet tips between liquid transfers. To avoid cross-contaminations, we recommend the use of aerosol-barrier pipet tips.
2. All centrifugation steps are carried out at room temperature.
3. When working with chemicals, always wear a suitable lab coat, disposable gloves, and protective goggles.
4. Discard gloves if they become contaminated.
5. Do not combine components of different kits unless the lot numbers are identical.
6. Avoid microbial contamination of the kit reagents.
7. To minimize the risk of infections from potentially infectious material, we recommend working under laminar air-flow until the samples are lysed.
8. This kit should only be used by trained personnel.

Important indications

Addition of Internal Controls (IC)

Using the RTP® Pathogen Kit protocol in combination with commercially available amplification systems may require the introduction of an internal control into the purification procedure. Internal control RNA or DNA should be added to the lysate. For optimal purification efficiency, internal control molecules should be longer than 100 nucleotides, as smaller molecules are not efficiently recovered. Refer to the manufacturer’s instructions in order to determine the optimal concentration.

Handling of RTA Spin Filter

Due to the sensitivity of viral DNA/ RNA amplification technologies, the following precautions are necessary when handling RTA Spin Filter to avoid cross-contamination between sample preparations

1. carefully apply the sample or solution to the RTA Spin Filter, pipet the sample into the filter without wetting the rim of the column
2. always change pipet tips between liquid transfers, we recommend the use of aerosol barrier pipet tips
3. avoid touching the RTA Spin Filter membrane with the pipet tip

Yield and quality of pathogen DNA/ RNA

Different amplification systems vary in efficiency depending on the total amount of nucleic acid present in the reaction. Eluates derived by this kit will contain Carrier-RNA, which will greatly exceed the amount of the isolated NA.

Yields of viral nucleic acids isolated from biological samples are usually low concentrated and therefore almost impossible to determine photometrically*.

*) Keep in mind that the Carrier-RNA (5 µg per 200 µl sample) will account for most of the present DNA/ RNA.

The kit is suitable for downstream analysis with NAT techniques, for examples qPCR, RT qPCR, LAMP, LCR. Diagnostic assays should be performed according to the manufacturer's instructions.

Quantitative RT-PCR is recommended for determination of viral RNA yield.

In Gel Electrophoresis and in Capillary Electrophoresis, DNA extracted with the provided kit looks like degraded cause the kit contains Carrier RNA, this is poly A RNA in fragments of 100 up to 1000 bases. The kit is not dedicated for applications using this kind of analysis.

Preparing buffers

1. Adjust the thermomixer to 37°C.
2. Heat heating blocks (e.g. thermomixer) to 65°C and 95 °C.
3. Warm up the needed amount of Elution Buffer R to 65°C, (60 - 200 µl Elution Buffer R are needed per sample).
4. Label the needed amount of 2.0 ml RTA Spin Filter Sets.
5. Label the needed amount of 1.5 ml Receiver Tubes (per sample: 1 Receiver Tube), add the needed amount of ethanol to the **Wash Buffer R1** and **R2** (see Kit Contents, page 3).

50 DNA/ RNA extractions:

Fill 30 ml 99.7% **Isopropanol** (molecular biologic grade) into the empty bottle
Add 20 ml 96-100% ethanol to the bottle **Wash Buffer R1**.
Add 48 ml 96-100% ethanol to each bottle **Wash Buffer R2**.
Mix thoroughly and always keep the bottle firmly closed!

250 DNA/ RNA extractions:

Fill 120 ml 99.7% **Isopropanol** (molecular biologic grade) into the empty bottle
Add 80 ml 96-100% ethanol to each bottle **Wash Buffer R1**.
Add 200 ml 96-100% ethanol to each bottle **Wash Buffer R2**.
Mix thoroughly and always keep the bottle firmly closed!

Important: Some specific cell culture media may inhibit lysis efficiency or downstream reactions.

For special comments on the diverse **sample preparation procedures** see page 14 - 16.

Equipment and reagents to be supplied by user

When working with chemicals, always wear a suitable lab coat, disposable gloves and protective goggles. For more information's, please consult the appropriate material safety data sheets (MSDS). (See our webpage: www.invitek-molecular.com)

- Microcentrifuge $\geq 11.000 \times g$ (≥ 11.000 rpm)
- Thermomixer (37°C - 95°C)
- ddH₂O
- Ethanol (96-100%)
- 2.0 ml reaction tubes (optional)
- Measuring cylinder (250 ml)
- Disposable gloves
- Pipet with tips
- 1.5 ml reaction tubes
- optional centrifuge for 15 or 50 ml
- Isopropanol*

*) The **RTP® Pathogen Kit** is validated with 2-Propanol; Rotipuran >99.7%, p.a., ACS, ISO (Order no. 6752) from **Carl Roth**.

*) **Possible suppliers for Isopropanol:**

Carl Roth
2-Propanol
Rotipuran >99.7%, p.a., ACS, ISO
Ordering No. 6752

Applichem
2-Propanol für die Molekularbiologie
Ordering No. A3928

Sigma
2-Propanol
Ordering No. 59304-1L-F

Sampling and storage of starting material

Please read the instructions carefully and conduct the prepared procedure.

Sampling and storage

For reproducible and high yields appropriate sample storage is essential. Yields may vary from sample to sample depending on factors such as sample age, kind of sample, transport and storage conditions.

Best results are obtained using freshly extracted samples. As long as the samples are not shock frosted with liquid nitrogen or are incubated with RNase inhibitors or denaturing reagents, the viral RNA is not secured. Therefore, it is essential, that samples are immediately flash frozen subsequent to the harvesting by using liquid nitrogen and are stored at -80°C. Viral RNA contained in such deep-frozen samples is stable for months. Viral RNA purification should be processed as soon as possible. Samples can also be stored in the dissolved Lysis Buffer in the Extraction Tube for 1 h at room temperature, overnight at 4°C, and for long-term storage at -80°C. Storage under deep frozen conditions is recommended.

Serum and plasma

After collection and centrifugation, serum, plasma, from blood (treated with anticoagulants like EDTA or citrate, but not with heparin) can be stored on ice for 1 - 2 hours, for short time samples may be stored at - 20°C. For long-term storage, we recommend freezing samples in aliquots at -80°C. Frozen plasma or serum samples must not be thawed more than once. Multiple thawing and freezing before isolating the viral DNA/ RNA should be avoided. It leads to denaturation and precipitation of proteins, resulting in reduced viral titers and therefore reduced yields of viral nucleic acids. In addition, cryoprecipitate formed during freeze-thawing could make problems. If cryoprecipitate is visible, they should be pelleted by centrifugation at app. 6.800 x g for 3 minutes. The cleared supernatant should be aspirated, without disturbing the pellet and processed immediately. This step will not reduce viral titers.

Swabs

The protocol works with prepared swabs as well as with dried swabs. The protocol has not been validated for isolation of DNA from swabs which are stored under special storage buffers of other providers.

Biopsy material/ tissue

Best results are obtained with fresh material or material that has been immediately frozen and stored at -20°C or -80°C. Repeated freezing and thawing of stored samples should be avoided, since this leads to reduced DNA size. Use of poor-quality starting material also leads to reduced length and influences yield of purified DNA. The amount of purified DNA from max.10 mg tissue sample depends on the nature of starting material. The thawing process could proceed, e.g. in the Extraction Tube.

Cultivated bacteria

Bacteria have to be pelleted after cultivation. Best results are obtained with fresh material or material that has been immediately frozen and stored at -20°C or -80°C. Repeated freezing and thawing of stored samples should be avoided, since this leads to reduced DNA size.

Urine

The bacteria may be pelleted while the supernatant is removed. Best results are obtained with fresh pelleted material or bacteria pellets that has been immediately frozen and stored at -20°C or -80°C. Repeated freezing and thawing of stored samples should be avoided, since this leads to reduced DNA size. The amount of purified DNA from max.15-50 ml urine depends from the included bacteria titre.

Stool samples

Best results are obtained with fresh material. Stool samples contain DNases and RNases which realize quickly DNA and RNA digestion and degradation. The sample may be stored at -80°C.

Cell culture supernatants

Best results are obtained with fresh material or material that has been immediately frozen and stored at -20°C or -80°C after winning of the cell culture supernatant. Repeated freezing and thawing of stored samples can influence the sensitivity.

Invitek Molecular will be released of its responsibilities if other sample materials than described in the Intended Use are processed or if the sample preparation protocols are changed or modified.

Volume adjustment of sample material

In every case the starting material has to be adjusted to a complete liquid volume of 400 µl.

Starting with a standard sample volume of 200 leads to an adjustment with 200 µl dd H₂O.

Other volumes have to be added if starting volumes differs from 200 µl have to be adjusted as described below:

Sample volume [µl] + Adjustment volume [µl] = 400 µl

One may add first the adjustment volume and then the sample to the Extraction Tube or in another way adjust the sample with ddH₂O or PBS to 400 µl and then add this mixture to the Extraction Tube

Restrictions:

- Sample volumes of plasma, serum or other protein containing liquids should not exceed 200 µl.
- Sample Volume of human blood should not exceed 50 µl.
- Pellets may be resuspended in 400 µl Resuspension Buffer and added directly.
- From samples without high protein contents like media one may use 400 µl directly.

Preparation of starting material

1. Extraction of NA from, cell-free fluids (like serum, plasma, urine, transport media)

200 µl of this type of sample can be processed directly without any pre-preparations following protocol 1. Please keep in mind that the first step in the equipment is premixing of samples. Samples have to be at least “pipetable”, mean the presence of clumps and other solid materials leads to clots and prevents a normal workflow of the process. We recommend strictly controlling samples for coagulation by mixing several times overhead before usage on the instrument.

Attention: *For concentration of bacteria from urine pelleting by centrifugation may be helpful, after pelleting please start with 8. Extraction of DNA from bacteria pellets*

Follow the protocol from the beginning

2. Extraction of NA from rinse liquid from swab samples

a) for gram positive bacteria

Place the swab into the **Extraction Tube**, add 400 µl **Resuspension Buffer R** and mix by stirring with the swab. Break or cut the swab and close the tube.

Incubate the sample in a thermomixer for 10 min at 37°C and at 65°C for 10 min (continuous shaking increases the lysis procedure, it is possible also to do a temperature ramp between the two incubation steps, step 37° must last at least 8 minutes, then the ramp, and afterwards the step 65° must last at least 8 minutes.

For difficult to lyse bacteria, we recommend an additional Lysis Step at 95°C for 10 min

Continue with step two of the protocol.

b) for gram negative bacteria

Place the swab into the **Extraction Tube**, add 400 µl **Resuspension Buffer R** and mix by stirring with the swab. Break or cut the swab and close the tube.

Incubate the sample in a thermomixer for 10 min 65°C for (continuously shaking increases the lysis procedure). Remove the swab.

Continue with step two of the protocol.

c) for viral DNA and/or RNA

Place the swab into the provided **Extraction Tube**, add 400 µl **Resuspension Buffer R** and mix by stirring with the swab. Break or cut the swab and close the tube.

Place the Extraction Tube into a thermomixer and incubate for 15 minutes at 65°C (under continuously shaking), which leads to higher sensitivity. After lysis time carefully squeeze the swab on the wall of the tube and discard the swab.

For difficult to lyse viruses we recommend an additional Lysis Step at 95°C for 10 min

Continue with step two of the protocol.

3. Extraction of viral NA from viscous samples like sputum, tracheal secretes or BAL

Transfer 200 µl of sample into a RNase/DNase-free tube and add 200 µl saturated Acetylcysteine (ACC) solution to the sample (ratio sample to buffer must be 1:1). Incubate the mixture for 10 min at 95°C to reduce the viscosity and transfer 400 µl from the mixture into the **Extraction Tube**.

Follow the protocol from the beginning.

4. Extraction of bacterial NA from tracheal secretes or BAL (e.g. from *Mycobacteria*)

Non viscous samples: Transfer 1 ml of tracheal secret or BAL into a RNase/DNase-free tube and centrifuge at 9.300 x *g* (10.000 rpm) for 3 min. Discard the supernatant without disturbing the bacterial pellet: Resuspend the bacterial pellet in 400 µl distilled water or RNase free water and transfer the sample into the **Extraction Tube**.

Follow the protocol from the beginning.

Bacterial DNA from viscous samples: Transfer 1 ml of tracheal secrete or BAL into a RNase- and DNase-free tube and add 1 ml saturated Acetylcysteine (ACC) solution to the sample (ratio sample to buffer must be 1:1)

Incubate the mixture for 10 min at 95°C to reduce the viscosity and centrifuge at 9.300 x *g* (10.000 rpm) for 3 min. Discard the supernatant without disturbing the bacterial pellet directly.

Resuspend the bacterial pellet in 400 µl distilled water or RNase free water and transfer it into the **Extraction Tube**.

Follow the protocol from the beginning.

5. Extraction of viral NA from supernatant of stool suspension

Transfer 100 µl/ 100 mg stool sample into a 2 ml tube and add 900 µl RNase-free Water. Vortex the sample for 30 s followed by a 1 min centrifugation step at 12.000 x *g* (13.000 rpm).

Transfer 400 µl virus containing supernatant into the **Extraction Tube** (prevent the aspiration of swimming particles).

Follow the protocol from the beginning.

6. Extraction of bacterial NA from supernatant of stool suspension

Transfer 100 µl/ 100 mg stool sample into a 2 ml tube and add 300 µl RNase-free Water. Vortex the sample for 30 s followed by a 30 s centrifugation step at 3.000 rpm. (1.000 x *g*)

Transfer 400 µl of the bacteria containing supernatant into the **Extraction Tube** (prevent the aspiration of swimming particles).

Follow the protocol from the beginning.

7. Extraction of total nucleic acids from tissue biopsies

Transfer 1 mg up to max. 10 mg of the tissue sample into the provided **Extraction Tube**. Add 400 µl of **Resuspension Buffer R**. Close the cap and vortex shortly. Place the **Extraction Tube** into a thermomixer and incubate under continuously shaking for 30 – 60 min minutes at 56°C.

Optional: Place the **Extraction Tube** into a thermomixer and incubate for 5 - 10 min at 95°C (continuous shaking increases the lysis efficiency) if very difficult to lyse virus or bacteria are used,

Lysis times may be increased if the lysis is not completely.

Important: A longer lysis time could reduce the final yield and the quality of some viral RNA species.

After lysis centrifuge the sample at max. speed for 1 minute to spin down unlysed material. Transfer the cleared supernatant completely into a 1.5 ml reaction tube (not provided).

Continue with step two of the protocol.

8. Extraction of DNA from bacterial pellets (up to 1 x 10⁹ bacteria cells)

a) for gram positive bacteria

Resuspend the pellet in 400 µl **Resuspension Buffer R** and transfer the sample into the **Extraction Tube** and vortex shortly.

Incubate the sample in a thermomixer for 10 min at 37°C and at 65°C for 10 min (continuous shaking increases the lysis procedure, it's possible also to do a temperature ramp between the two incubation steps, step 37° must last at least 8 minutes, then the ramp, and afterwards the step 65° must last at least 8 minutes.

For difficult to lyse bacteria, we recommend an additional Lysis Step at 95°C for 10 min

Continue with step two of the protocol

b) for gram negative bacteria

Resuspend the pellet in 400 µl **Resuspension Buffer R** and transfer the sample into the **Extraction Tube** and vortex shortly. Incubate the sample in a thermomixer at 65°C for 10 min (continuous shaking increases the lysis procedure).

For difficult to lyse bacteria, we recommend an additional Lysis Step at 95°C for 10 min

Continue with step two of the protocol

Instructions

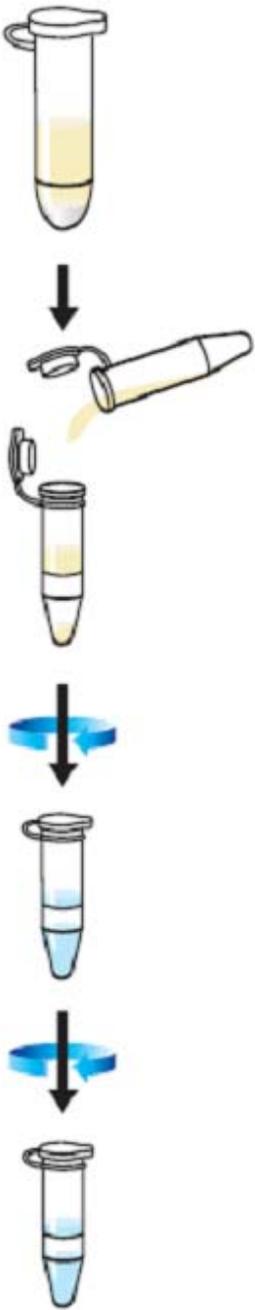
Note: The DNA/ RNA can also be eluted with a lower (but not lower than 40 µl) or a higher volume of **Elution Buffer R** (depends on the expected yield or needed concentration of the DNA/ RNA).
The eluate contains viral DNA and/ or viral RNA as well as sometimes genomic DNA.

Important After extraction, place the **Elution Tube** on ice. For long time storage, place the nucleic acids at –20°C or –80°C.

Note: The centrifugation steps were made with the **Centrifuge 5415 D from Eppendorf**. The indicated **rpm amounts** are referring to this centrifuge.

Scheme of the RTP® Pathogen Kit

Please work quickly and perform all extraction steps at room temperature (RT)!

	<p>Please read the protocols carefully prior to the start of the preparation procedure!</p> <p>Add 400 µl of volume adjusted sample material optional: incubate for 10 min at 37°C in a thermomixer (only gram-positive bacteria)</p> <p>incubate for 10 – 15 min. at 65°C in a thermomixer optional: incubate for 5 - 10 min. at 95°C in a thermomixer</p> <p>If you want to add nucleic acids for extraction control, please add at this point</p> <p>for optimal binding conditions add 400 µl Binding Solution and mix the sample completely by pipetting up and down or by vortexing</p> <p>transfer the sample on the RTA Spin Filter, incubate for 1 min. centrifuge for 2 min. at 11.000 x g (11.000 rpm)</p> <p>discard the flow-through with the RTA Receiver Tube put the RTA Spin Filter in a new RTA Receiver Tube</p> <p>pipet 500 µl Wash Buffer R1 onto the RTA Spin Filter centrifuge 1 min. at 11.000 x g (11.000 rpm) discard the flow-through and the RTA Receiver Tube transfer the RTA Spin Filter into a new RTA Receiver Tube</p> <p>pipet 700 µl Wash Buffer R2 onto the RTA Spin Filter centrifuge for 1 min. at 11.000 x g (11.000 rpm) discard the flow-through and the RTA Receiver Tube transfer the RTA Spin Filter into a new RTA Receiver Tube</p> <p>to eliminate any traces of ethanol, centrifuge again for 4 min at maximum speed discard the RTA Receiver Tube</p> <p>transfer the RTA Spin Filter into a RNase free 1.5 ml Elution Tube pipet 60 µl of Elution Buffer R (preheated to 65°C) directly onto the membrane of the RTA Spin Filter incubate for 3 min at RT centrifuge for 1 min at 11.000 x g (11.000 rpm)</p> <p>discard the RTA Spin Filter and place the eluted pathogen NA immediately on ice!</p>
--	---

Protocol: Simultaneous isolation of total nucleic acids (DNA and RNA) from all liquid samples

Please read the protocols carefully prior to the start of the preparation procedure!

Important Note: Preheat the needed amount of **Elution Buffer R** to 65°C for the final elution step. The protocol has been optimized for the isolation of total nucleic acids from 200 µl sample volume. For samples which have a smaller volume than 200 µl please fill up to a total volume of 400 µl with ddH₂O, please check also page 14 Volume adjustment of sample material

1. Sample Lysis

Add 400 µl of volume adjusted sample material to the provided **Extraction Tube**. Close the cap and vortex shortly.
Place the Extraction Tubes L into a thermomixer and incubate under continuously shaking for 15 minutes at 65°C.

For difficult to lyse bacteria we recommend an additional Lysis step at 95°C for 10 min

If you want to add nucleic acids for extraction control, please add at this point

2. Binding of the DNA and RNA

Add 400 µl **Binding Solution** to the provided **Extraction Tube** and mix the sample completely by pipetting up and down or by vortexing. Transfer the sample into the RTA Spin Filter Set. Close the cap and centrifuge for 2 minutes at 11.000 x g (11.000 rpm). Discard the RTA Receiver Tube with filtrate and place the RTA Spin Filter into a new RTA Receiver Tube.

3. First Washing of the RTA Spin Filter

Add 500 µl **Wash Buffer R1** to the RTA Spin Filter and centrifuge at 11.000 x g (11.000 rpm) for 1 minute. Discard the RTA Receiver Tube with filtrate and place the RTA Spin Filter into a new RTA Receiver Tube.

4. Second Washing of the RTA Spin Filter

Add 700 µl **Wash Buffer R2** to the RTA Spin Filter and centrifuge at 11.000 x g (11.000 rpm) for 1 minute. Discard the RTA Receiver Tube with filtrate and place the RTA Spin Filter into a new RTA Receiver Tube.

Remove the residual ethanol by final centrifugation for 4 min at maximum speed.
Discard the RTA Receiver Tube with filtrate.

5. Elution of the DNA/ RNA

Place the Spin Filter into a 1.5 ml Elution Tube and add 60 µl of the **Elution Buffer R** (preheated to 65°C) directly onto the RTA Spin Filter surface.
Incubate for 3 minutes at RT and centrifuge at 11.000 x g (11.000 rpm) for 1 minute.

Troubleshooting

Problem	Probable cause	Comments and suggestions
clogged RTA Spin Filter	insufficient lysis, homogenization and/or too much starting material	increase lysis time increase g-force and/ or centrifugation time reduce amount of starting material all centrifugation steps should be conducted at room temperatures
low amount of extracted DNA/ RNA	insufficient lysis incomplete elution insufficient mixing of the sample with Binding Solution incomplete removal of cell culture medium	increase lysis time reduce amount of starting material; overloading of RTA Spin Filter reduces yield prolong the incubation time with prewarmed Elution Buffer R to 5 -10 min. do the elution steps twice. take higher volume of Elution Buffer R mix sample sufficient by pipetting up and down with Binding Solution prior to transfer the sample onto the RTA Spin Filter make sure that the cell culture medium is complete removed after the cell harvest
low concentration of extracted DNA/ RNA	too much Elution Buffer R incorrect storage of starting material	elute the DNA/ RNA twice with lower volume of Elution Buffer R ensure that the storage of starting material was correct; avoid thawing of the material
DNA/ RNA does not perform well in downstream-applications (e.g. RT-PCR or PCR)	ethanol carryover during elution salt carryover during elution	increase g-force or centrifugation time when drying the RTA Spin Filter. ensure that the Wash Buffers are at room temperature check up the Wash Buffers for salt precipitates. If there are any precipitates, dissolve these precipitates by carefully warming

Appendix

General notes on handling DNA

Nature of DNA

The length and delicate physical nature of DNA require careful handling to avoid damage due to shearing and enzymatic degradation. Other conditions that affect the integrity and stability of DNA include acidic and alkaline environments, high temperature, and UV irradiation. Careful isolation and handling of high molecular weight DNA is necessary to ensure it will work well in various downstream applications. Damaged DNA could perform poorly in applications such as genomic Southern blotting, long-template PCR, and construction of cosmid libraries.

Handling fresh and stored material before the extraction of DNA

For the isolation of genomic DNA from cells or tissues, use either fresh samples or samples that have been quickly frozen in liquid nitrogen and stored at -70°C . This procedure minimizes degradation of crude DNA by limiting the activity of endogenous nucleases.

Storage of DNA

Store DNA at $+2$ to $+8^{\circ}\text{C}$. Storing genomic DNA at -15 to -25°C can cause shearing of DNA, particularly if the DNA is exposed to repeated freeze-thaw cycles. Plasmid DNA and other small circular DNAs can be stored at $+2$ to $+8^{\circ}\text{C}$ or at -15 to -25°C .

Drying, dissolving and pipetting DNA

Avoid overdrying genomic DNA after ethanol precipitation. It is better to let it air dry than to use a vacuum, although vacuum drying can be used with caution. Plasmid DNA and other small circular DNAs can be vacuum-dried.

To help dissolve the DNA, carefully invert the tubes several times after adding buffer and tap the tube gently on the side. Alternatively let the DNA stand in buffer overnight at $+2$ to $+8^{\circ}\text{C}$. Minimize vortexing of genomic DNA since this can cause shearing.

Avoid vigorous pipetting. Pipetting genomic DNA through small tip openings causes shearing or nicking. One way to decrease shearing of genomic DNA is to use special tips that have wide openings designed for pipetting genomic DNA. Regular pipette tips pose no problem for plasmid DNA and other small.

Quantification

Quantification of DNA and RNA from this assay must be done by means of amplification / hybridization techniques-based measurement. Other methods like staining and absorption measurements will be disturbed by the included Carrier Nucleic Acids as well as DNA or RNA, which is co purified.

General notes on handling RNA

RNA is far less stable than DNA. It is very sensitive to degradation by endogenous RNases in the biological material and exogenous RNases which are permanently present everywhere in the lab. To achieve satisfactory qualitative and quantitative results in RNA preparations, contaminations with exogenous RNases has to be reduced as much as possible. Avoid handling bacterial cultures, cell cultures, or other biological sources of RNases in the same lab where the RNA purification is to be carried out.

All glassware should be treated before use to ensure that it is RNase free. Glassware should be cleaned with detergent, thoroughly rinsed and oven baked at 240° C for four or more hours before use. Autoclaving alone will not completely inactivate many RNases. Oven baking will both inactivate RNases and ensure that no other nucleic acids (such as Plasmid DNA) are present on the surface of the glassware. You can also clean glassware with 0.1% DEPC (diethyl pyrocarbonate). The glassware must stand 12 hours at 37° C and then be autoclaved or heated to 100°C for 15 min to remove residual DEPC.

- Electrophoresis tanks should be cleaned with detergent solution (e.g. 0.5 % SDS), thoroughly rinsed with RNase free water, and then rinsed with ethanol and allowed to dry.
- Non-disposable plasticware should be treated before use to ensure that it is RNase free. Plastic ware should be thoroughly rinsed with 0.1 M NaOH, 1 mM EDTA followed by RNase free water. You can also take chloroform-resistant plastic ware rinsed with chloroform to inactivate RNases.
- All buffers must be prepared with RNase free ddH₂O.
- Change gloves frequently and keep tubes closed.
- Reduce the preparation time as much as possible.
- Use only sterile, disposable polypropylene tubes throughout the procedure. (These tubes are generally RNase free.)
- Keep isolated RNA on ice.
- This kit should only be used by personnel trained in *in vitro* diagnostic laboratory practice.

Storage of RNA

Purified RNA can be stored –80°C and is stable for months and years e.g. precipitated and stored in 70% ethanol.

Quantification

Quantification of DNA and RNA from this assay must be done by means of qPCR or Reverse Transcriptase qPCR. All other methods will be disturbed by the included Carrier Nucleic Acids as well as DNA or RNA, which is co purified.

Ordering information

Product	Package Size	Catalogue No.
RTP® Pathogen Kit	50 preparations	1040500200
RTP® Pathogen Kit	250 preparations	1040500300

Possible suppliers for Isopropanol:

Carl Roth

2-Propanol
Rotipuran >99.7%, p.a., ACS, ISO
Ordering no. 6752

Applichem

2-Propanol für die Molekularbiologie
Ordering no. A3928

Sigma

2-Propanol
Ordering no. 59304-1L-F

Possible suppliers for Centrifuges:

Eppendorf AG

22331 Hamburg, Germany
Phone: +49 (0) 40 53801 0
Fax: +49 (0) 40 53801 556
E-Mail: eppendorf@eppendorf.com
www.eppendorf.com

SIGMA Laborzentrifugen GmbH

37507 Osterode am Harz, Germany
Phone: +49-5522-5007-0
Fax: +49-5522-5007-12
E-Mail: info@sigma-zentrifugen.de
www.sigma-zentrifugen.de

INVITEK
Molecular

Invitek Molecular GmbH
Robert-Rössle-Str. 10
13125 Berlin, Germany

Phone: +49 30 9489 2908
Fax: +49 30 9489 3795
info@invitek-molecular.com

www.invitek-molecular.com

0061040500 V-01-2020